

Application Instructions



Bio Skin® supports should be applied to dry skin only. Oils, lotions and/or perspiration activate "SkinLok™" making correctly sized supports seem small and difficult to apply. This response to moisture should not cause confusion with respect to proper sizing. Always dry the body segment thoroughly prior to application.

Take the support with both hands and stretch the top and bottom of the brace two to three times. This helps break-in the material.

1. Position silicone buttress over patellar tendon. Wrap ends, securing the hook on top of opposite end, and secure to a snug fit.



2. Attach the strap to the support with the buckle on either side of the silicone pad. Wrap the elastic strap around the back of the leg. Insert the strap through the buckle and tighten until comfortably snug. Then attach the Velcro®.



3. The elastic strap is tightened to comfort level.



Easy Care Instructions

Wash Bio Skin® regularly to prevent body salt and oil build-up that naturally occurs after the product has been worn a few times. Attach the hook to the loop to prevent lint build-up. This will give your Bio Skin® a longer life span.



Hand Wash



Hang Dry



Do Not Bleach



Do Not Dry Clean

Cropper Medical, Inc.
custserv@bioskin.com
800.541.2455
www.bioskin.com

