

# Bio Skin® TriLok™ Ankle Control System

## Application Instructions

*Looking down at your foot*

### 1 Undersleeve



- Pull the sleeve on over foot.
- Secure both 1½" straps with grey tabs.
- Make sure the top strap is tight.


**Note:** Leave collar with Bio Skin® logo up until step 4.

### 2 FootLok Strap™




- The white FootLok Strap™ **MUST BE APPLIED** for maximum stability.
- Follow the colored instructions printed on the FootLok Strap™. Start at the narrow end, noting Right or Left foot.

### 3 Stirrup Strap (Marked with the TriLok™ logo)

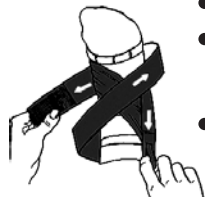

- Open strap by pulling the finger loops on each end.
- With TriLok™ logo facing out, attach strap with logo over achilles tendon.
- Cross each strap over the top of the foot then cross both under the foot. Each end is pulled up snugly and attached to the sides of the ankle.
- Keep strap attached to brace after use.

### 4 Collar

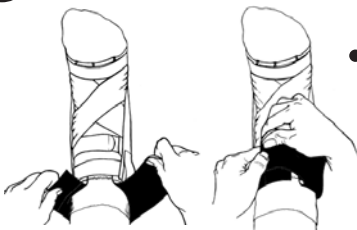

- Roll collar down and secure.



**TriLok™**

### Easy Care Instructions

Wash Bio Skin® regularly to prevent body salt and oil build-up that naturally occurs after the product has been worn a few times. Attach the hook to the loop to prevent lint build-up. This will give your Bio Skin® a longer life span.



Hand Wash



Hang Dry



Do Not Bleach



Do Not Dry Clean

**Cropper Medical, Inc.**  
custserv@bioskin.com  
800.541.2455  
**www.bioskin.com**

  
**Bio Skin®**