

Thigh Skin™

Congratulations! you have purchased the most sophisticated compression support available. The unique features of Bio Skin® offer protection while providing function, comfort, and style.

Bio Skin® offers a broad spectrum of advantages over conventional vinyl and neoprene supports. The physical properties of this new material allow compression that conforms to the dynamics of the joint like no other support. Our products are breathable, light, thin, and strong.

Bio Skin® products wick away moisture and are easy to apply and adjust.



Application Instructions

Bio Skin® supports should be applied to dry skin only. Oils, lotions and/or perspiration activate "SkinLok™" making correctly sized supports seem small and difficult to apply. This response to moisture should not cause confusion with respect to proper sizing. Always dry the body segment thoroughly prior to application.

Take the support with both hands and stretch the top and bottom of the brace two to three times. This helps break-in the material.



Pull Thigh Skin™ on to fit as high on the thigh as possible. The sleeve should fit very tight.



Applying the Cinch Strap

Pull Thigh Skin™ on to fit as high on the thigh as possible. The adjustable cinch strap can be attached to the fuzzy loop along the total length of the sleeve. The strap should be placed over the painful area and tightened to comfort.

Easy Care Instructions

Wash Bio Skin® regularly to prevent body salt and oil build-up that naturally occurs after the product has been worn a few times. Attach the hook to the loop to prevent lint build-up. This will give your Bio Skin® a longer life span.



Hand Wash



Hang Dry



Do Not Bleach



Do Not Dry Clean

