

Calf Skin™

Congratulations! you have purchased the most sophisticated compression support available. The unique features of Bio Skin® offer protection while providing function, comfort, and style.

Bio Skin® offers a broad spectrum of advantages over conventional vinyl and neoprene supports. The physical properties of this new material allow compression that conforms to the dynamics of the joint like no other support. Our products are breathable, light, thin, and strong.

Bio Skin® products wick away moisture and are easy to apply and adjust.



Application Instructions

Take the support with both hands and stretch the top and bottom of the support two to three times. This helps to break-in the material.

Pull sleeve on over the heel and up the calf to fit from below the knee to just above the ankle.

Easy Care Instructions

Wash Bio Skin® regularly to prevent body salt and oil build-up that naturally occurs after the product has been worn a few times. Attach the hook to the loop to prevent lint build-up. This will give your Bio Skin® a longer life span.



Hand Wash



Hang Dry



Do Not Bleach



Do Not Dry Clean

