

Shinsplints Wrap

Congratulations! you have purchased the most sophisticated compression support available. The unique features of Bio Skin® offer protection while providing function, comfort, and style.

Bio Skin® offers a broad spectrum of advantages over conventional vinyl and neoprene supports. The physical properties of this new material allow compression that conforms to the dynamics of the joint like no other support. Our products are breathable, light, thin, and strong.

Bio Skin® products wick away moisture and are easy to apply and adjust.



Application Instructions



1. Place sewn-in pad (end opposite the label) over painful area on the front of the shin.



2. While holding the pad end in place, wrap the support around the leg and secure the first hook strip onto the wrap.



3. Continue wrapping and tension the wrap to comfort and fasten end to secure.

Easy Care Instructions

Wash Bio Skin® regularly to prevent body salt and oil build-up that naturally occurs after the product has been worn a few times. Attach the hook to the loop to prevent lint build-up. This will give your Bio Skin® a longer life span.



Hand Wash



Hang Dry



Do Not Bleach



Do Not Dry Clean

