


Application Instructions



Bio Skin® supports should be applied to dry skin only. Oils, lotions and/or perspiration activate "SkinLok™" making correctly sized supports seem small and difficult to apply. This response to moisture should not cause confusion with respect to proper sizing. Always dry the body segment thoroughly prior to application.

1. Slip thumb into the hole so the narrow area fits into the thumb crotch.

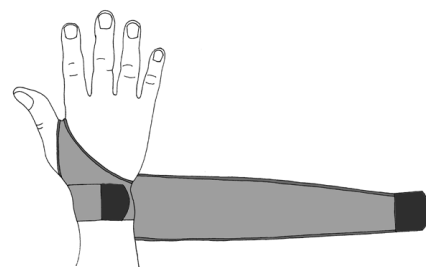

2. Wrap short strap under long strap around wrist and attach.



3. Wrap long strap around wrist and attach.

Easy Care Instructions

Wash Bio Skin® regularly to prevent body salt and oil build-up that naturally occurs after the product has been worn a few times. Attach the hook to the loop to prevent lint build-up. This will give your Bio Skin® a longer life span.



Hand Wash



Hang Dry



Do Not Bleach



Do Not Dry Clean

Cropper Medical, Inc.
custserv@bioskin.com
800.541.2455
www.bioskin.com

